


# Quick Setup Guide \*

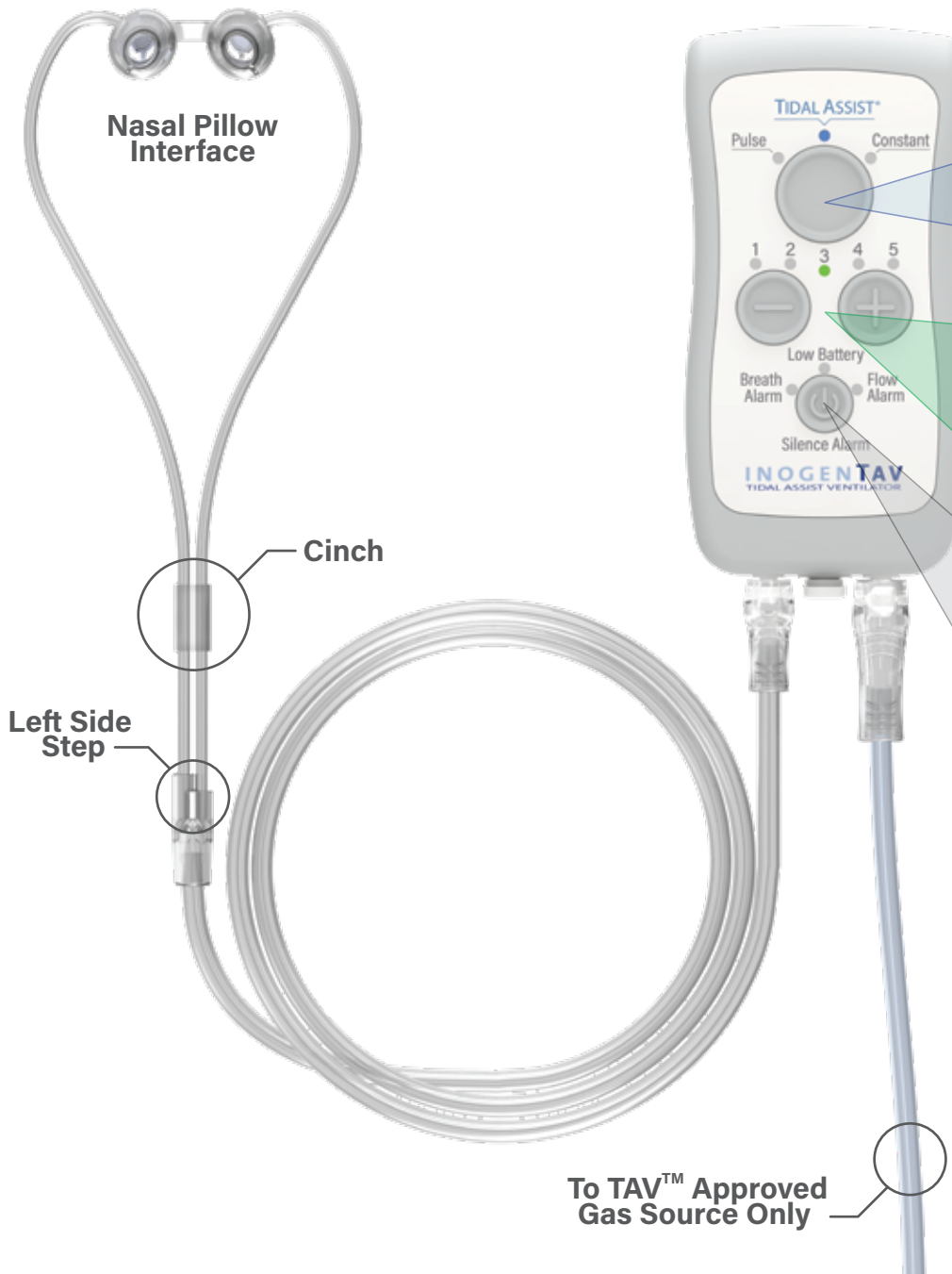

**Mode** Page 10-13

Tidal Assist

Pulse Constant

**Level** Page 10-13

DECREASE FLOW

INCREASE FLOW


**Power / Silence Alarm** Page 10-13

Hold 3 seconds  
On / Off

Press briefly to  
Silence Alarm

\* Reference Instructions For Use for additional information  
96-09783-00-01 C, TAV QUICK REFERENCE GUIDE

# Quick Troubleshooting Guide \*

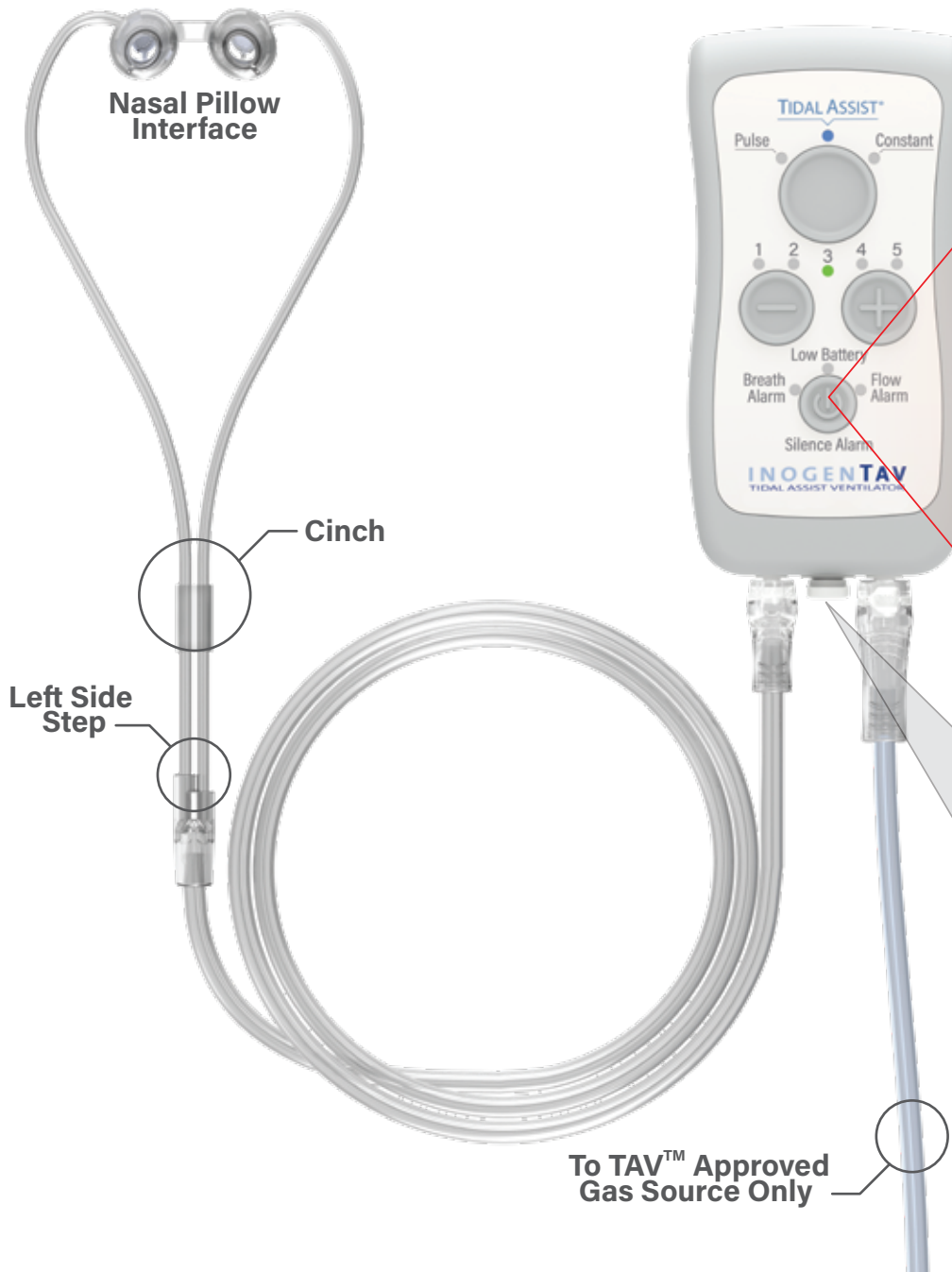

## Alarms



1. Check patient
2. Check nasal pillow interface fit
3. Check tubing for kinks and leaks
4. Check all connections
5. Turn Device off and then back on



Battery low - replace battery



1. Check tubing for kinks and leaks
2. Check connections
3. Check gas source
4. Check regulator adaptor
5. Turn Device off and then back on



1. Check Bypass Valve is closed
2. Turn Device off and then back on

*Note: if alarms cannot be cleared, turn off device and call customer support. Use Bypass Valve if needed.*

## Bypass Valve (2LPM constant)

Power off



Hold 3 seconds



Pull

\* Reference Instructions For Use for additional information