


## HIGH-FLO Q&A

**Q. How do I know if I am using the correct length needle?**

**A.** Needle length should be determined according to how much sub-q tissue you have. 9 & 12 mm needles tend to work well with average sized patients; while 6 mm needles may be needed for very thin adults & children. Similarly, larger patients may require 14 mm needles.

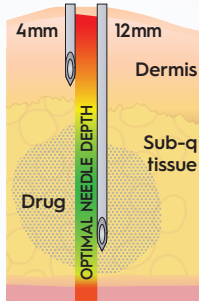


**Q. Why are they called "HlgH-Flo"?**

**A.** Made for sub-q, 26 & 24 gauge HlgH-Flo (high performance flow) sets offer faster infusion rates than typical 27 gauge needle sets.

*Note: If you are transitioning from 27 gauge, your provider might give you a different F-numbered Precision tubing set to keep your infusion time similar.*

**Q. Why are they called "safety" needle sets?**

**A.** The butterfly wings 'snap shut' at the ends, to cover the needle after use. This feature is designed to help prevent accidental needle-stick injuries.

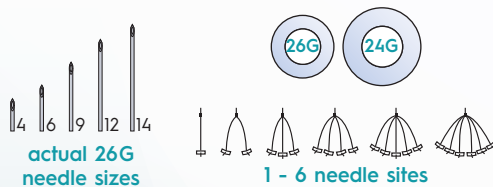


**Q. What needle set options do you offer?**

**A.** HlgH-Flo needles come in various set options to fit patients' needs. Infuse with 1, 2, 3, 4, 5, or 6 sites. Available needle lengths are 4, 6, 9, 12 & 14 mm in 26 & 24\* gauge.

*\*24 gauge sets are generally used for large volume infusions. Not available in 4 mm.*

HlgH-Flo needles are designed as part of a complete integrated infusion system, with options for every patient.

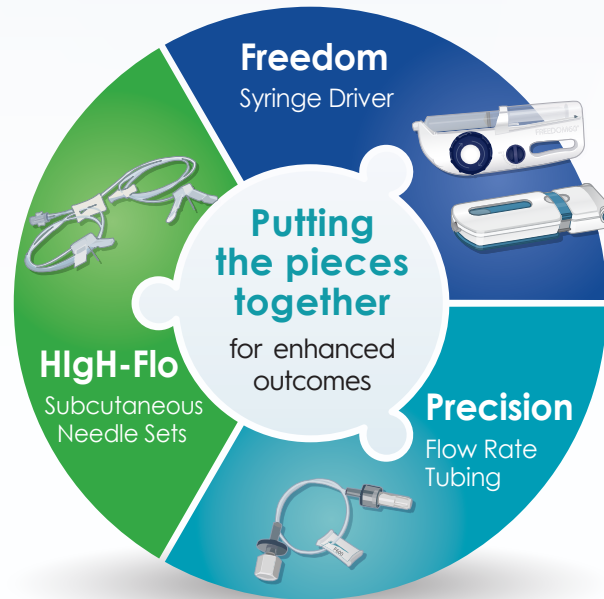

## GET TO KNOW

# HlgH-Flo™

Subcutaneous Safety Needle Sets

“It is easy to switch and I noticed improvements in my infusions right away.”

-Joanna T., SClg Patient

### PATIENT RESOURCES:

- [facebook.com/korumedical](https://facebook.com/korumedical)
- [twitter.com/korumedical](https://twitter.com/korumedical)
- [instagram.com/korumedical](https://instagram.com/korumedical)
- [bit.ly/koruvids](https://bit.ly/koruvids)
- [info@korumedical.com](mailto:info@korumedical.com)
- [Connections Blog: korumedical.tumblr.com](https://korumedical.tumblr.com)
- [FREEDOM60® Training Video: bit.ly/f60video](https://bit.ly/f60video)
- [FreedomEdge® Training Video: bit.ly/fredgevideo](https://bit.ly/fredgevideo)

**KORU™**  
MEDICAL SYSTEMS | 845-469-2042  
korumedical.com

The FREEDOM60® and FreedomEdge® Syringe Infusion Systems, Precision Flow Rate Tubing™ and HlgH-Flo Subcutaneous Safety Needle Sets™ are registered trademarks of KORU Medical Systems and are compliant with Medical Device Directive 93/42/EEC. KORU Medical Systems is ISO 13485 certified. © 2019 KORU Medical Systems; All Rights Reserved. HFL\_BR\_GetToKnow\_03-2046v1



## FEATURE HIGHLIGHTS



HighFlo needle incision



other brand incision




Minimize tissue damage, scarring and local site reactions with the backcut needle tip design.

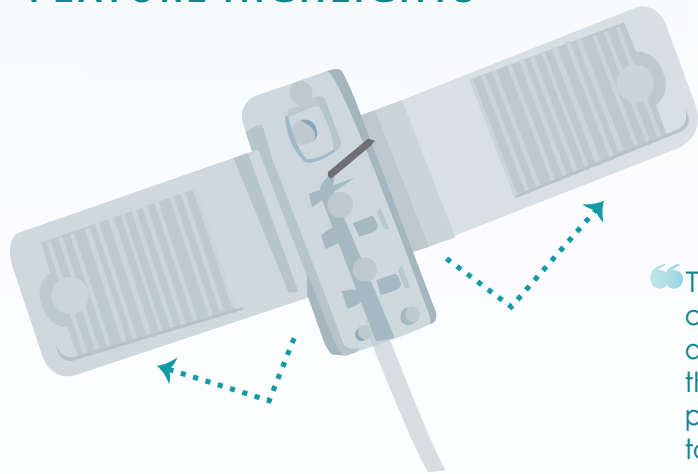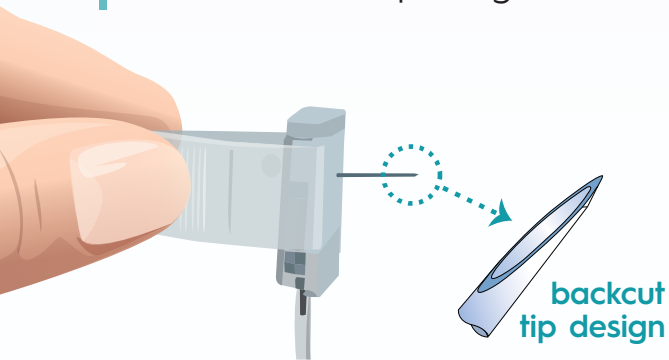

Needles stay in place after insertion, without tenting, due to the flexible, hinged butterfly wings.

“The butterfly wings on HighFlo seem a bit broader than those of my previous product. This helps to secure the shorter 9mm needle in place so it doesn't dislodge during infusion.”

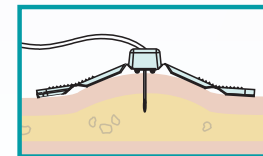

- Maureen H., SClg Patient

Efficient & practical tubing length:

- Less likely to tangle
- Faster flow rates




Protect against accidental needle stick injuries with the safety closure feature.

“I've never had another subcutaneous needle enter my skin so smoothly & painlessly, and exit with such ease. The comfort of HighFlo is unparalleled, and I've tried them all!”

- Dave K., SClg Patient & SP Sales Rep.

Custom luer design provides even flow to every site.

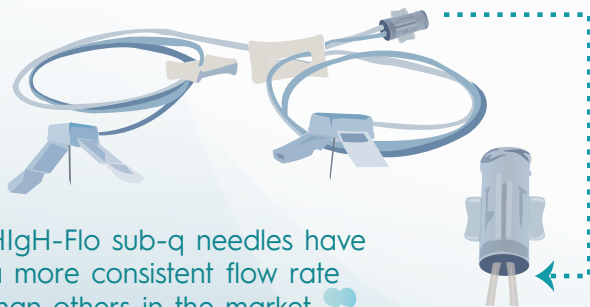

“HighFlo sub-q needles have a more consistent flow rate than others in the market.”

- Gene B., RPh

“The HighFlo needle tubing length is shorter than my previous product, and I like this... I feel more secure with the shorter tubing.”

- Maureen H., SClg Patient

**Note:** 24" Extension set available, for patients who want longer length.

“The higher-quality 3M Tegaderm™ dressing that is provided speaks to the way they care about their customers... a pleasant surprise.”

- Ted S., SClg Patient

