- EN** Subcutaneous Safety Needle Set - Instructions
- CS** Souprava subkutánních bezpečnostních jehel - Pokyny
- DA** Subkutan sikkerhedskanylesæt - Vejledning
- DE** Set mit subkutanen Sicherheitskanülen - Gebrauchsanleitung
- ES** Juego de agujas de seguridad subcutáneas - Instrucciones
- FI** Ihonalainen turvaneulapakkaus - Ohjeet
- FR** Kit d'aiguilles de sécurité sous-cutanées - Instructions
- HE** ערכת מחט בטוחות תת-עורית - תוארוה
- HU** Steril csővezeték-készlet - Útmutatás
- IT** Set di aghi di sicurezza sottocutanei - Istruzioni
- NO** Subkutan sikkerhetskanylesett - Instruksjoner
- PT** Agulha subcutânea com sistema de segurança - Instruções
- RU** Набор безопасных игл для подкожных инъекций - Инструкции
- SV** Infusionsset med säkerhetskanyl - Instruktioner
- TR** Subkutan Korumalı İğne Seti - Talimat

Store at room temperature.

Caution

Consult
Instructions
For Use

Sterilized Using Irradiation

Prescription
Only

Do Not
Reuse

Do Not
Resterilize

Do Not Use
if Package
is Damaged

Latex Free

Manufacturer

KORU Medical Systems
24 Carpenter Road
Chester, NY 10918 USA
800-624-9600 toll free
845-469-2042 local


European
Representative

MedPass International
95 bis Boulevard Pereire
75017 Paris, France
+33 (0)1 42 12 83 30

CE 2797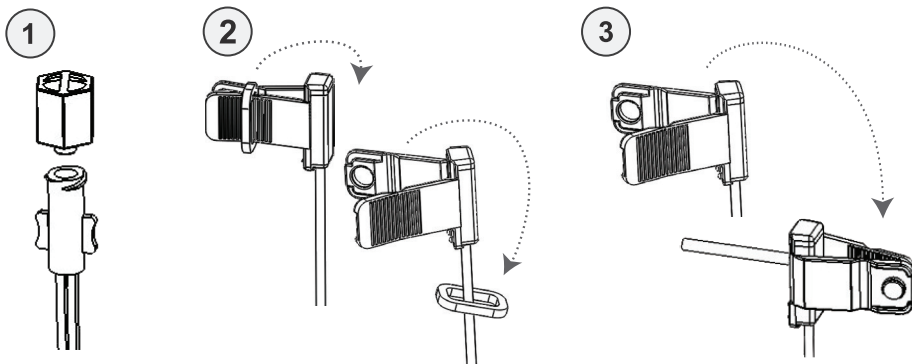

EN Indications for use: KORU Subcutaneous Needle Sets are intended for the subcutaneous infusion of prescribed liquid medicines from an external infusion pump or syringe.

⚠ Caution:

- U.S. Federal Law restricts this device to sale by or on the order of a physician.
- Use subcutaneous needle sets only for the patient in which the device is prescribed and only for its intended use.
- Subcutaneous needle sets are only sterile and non-pyrogenic if the packaging is unopened and undamaged. Do not use a needle set from an open or damaged package.
- Subcutaneous needle sets are single use items which should be disposed of after use. Please contact your healthcare provider for available sharps disposal containers.
- Do not clean or re-sterilize needle sets.
- Be sure that the needle guard is removed before insertion.
- Assure that the needle is not bent prior to insertion.
- Never prime the needle set or attempt to free a clogged tube while the needle is inserted. You may accidentally inject too much medication.
- When infusing through multiple needles, over infusion may occur to remaining sites if one or more of the needles are occluded.
- Inaccurate medication delivery, infection and/or site irritation may result from improper needle insertion and maintenance of the infusion site.
- Check the infusion set periodically to ensure that the needle remains firmly in place. If the needle is not in place, remove it and use a new needle set.
- When using needle sets for the first time, do so in the presence of a healthcare provider.

Instructions for use:

1. Remove the end cap from the female luer and connect the luer to the source of infusion.
2. Remove the wings closure band to separate each wing.
3. Bring the wings together and hold them between two fingers so that the needle guard is exposed.
4. Remove the needle guard and insert each needle into the subcutaneous tissue; cover with adhesive dressing.
5. When complete, remove adhesive and pull each needle out by bringing the wings together Snap closed to capture the needle and discard.

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HlgH-Flo Subcutaneous Safety Needle Sets™ are a trademark of KORU Medical Systems and are compliant with Medical Device Directive 93/42/EEC. KORU Medical Systems is ISO 13485 certified. ©2020 KORU Medical Systems; All Rights Reserved. 342102vH