


precision

FLOW RATE CONTROLLER™

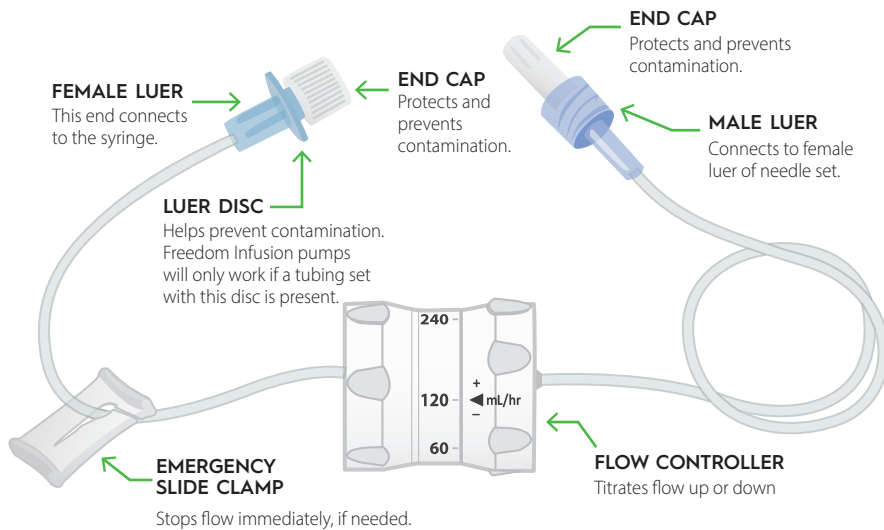

Ordering Information - SKU: F3CP (box of 10)

The Flow Rate Controller of Choice:

- Deliver 3 centipoise viscosity drugs up to 300ml/site/hr
- Safe and accurate flow control in accordance with drug manufacturers' Package Insert (PI)
- Empowers patient to do self-treatment with easy-to-follow product training
- Consistent delivery when used with 24g High-Flo Subcutaneous Safety Needle Sets™
- Constant pressure of Freedom DynEQ™ infusion system achieves repeatable and reliable infusions
- Titrate administrations up or down
- Integral flow rate safety feature
- Fewer SKUs in inventory


For use with 3cP drug delivery only.

Indications For Use

The Precision Flow Rate Controller™ is intended for use in the administration of 3 centipoise viscosity infusion fluids to be delivered to the patient in a precise manner over a specified period of time. Use only with KORU Medical Systems' Freedom Syringe Infusion Pumps.


How to use Precision Flow Rate Controller™:

1 Set Controller

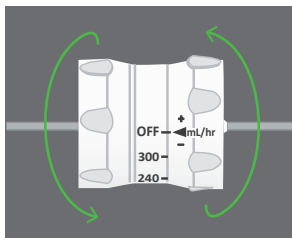


From OFF position, rotate controller to the setting determined by your healthcare professional.

2 Begin Infusion

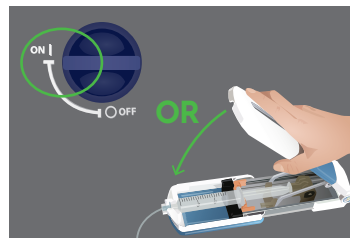


Begin infusion by turning pump ON (if using the FREEDOM60®) or by closing the top lid (if using the FreedomEdge®).

3 Set Rate

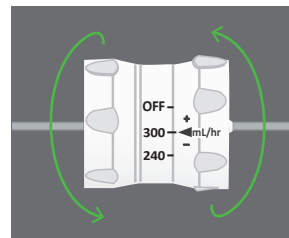

Select flow rate as directed.

4 Emergency

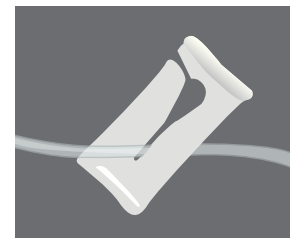

To immediately stop flow, in an emergency, pull the tubing through the slide clamp.

Other Products:



FREEDOM60® Syringe Infusion System
for larger infusion doses, 60 ml syringe



FreedomEdge® Syringe Infusion System
for smaller infusion doses, 20 ml and 30 ml syringes



Precision Flow Rate Tubing™
for flow rates from 0,5 to 2,400 ml/hr



High-Flo Subcutaneous Safety Needle Sets™
available in 24G and 26G, 4 mm-14 mm lengths and 1-6 needle leg sets



24" Extension Tubing Set (FEXT)
for those who need a longer length