
precision

FLOW RATE CONTROLLER™

Precision Flow Rate Controller™ Instructions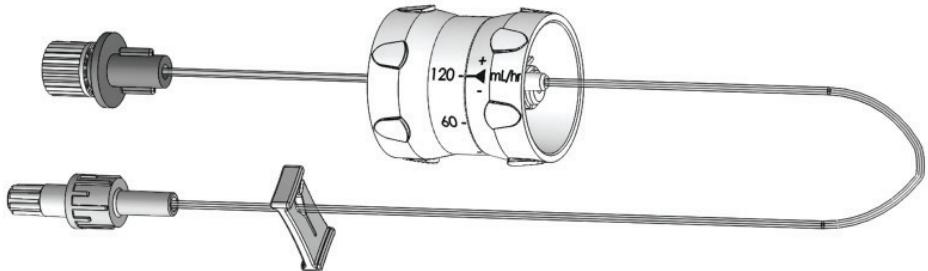

EN Instructions for Using the Precision Flow Rate Controller™

DA Brugsanvisning til Precision Flow Rate Controller™

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KORU
MEDICAL SYSTEMS

1. Warnings and Cautions:

- a. Use the Precision Flow Rate Controller™ only for the patient for whom the device is prescribed and only for its intended use.
- b. Before use, carefully inspect the flow rate controller package. Do not use if the package is opened or damaged.
- c. Do not use if the device is damaged or broken.
- d. Do not re-sterilize the controller and do not re-use.
- e. The flow controller is recommended to be stored at room temperature (approximately 16-30°C or 61-86°F).
- f. The flow controller should only be used with infusion pumps made by KORU Medical Systems.
- g. Do not infuse until all the air has been purged from the controller. See 4.2.
- h. Use only with 24 gauge HlgH-Flo Subcutaneous Safety Needle Sets™.
- i. Contact your healthcare provider if you do not understand settings or instructions.
- j. Save instructions for future use.

2. General Description

The Precision Flow Rate Controller™ is a component of and a device used to establish the flow rate of medications being infused with KORU Infusion Pumps. The Precision Flow Rate Controller™ performs the same function as Precision Flow Rate Tubing™ cleared for the Freedom60 Infusion System in premarket notification K933652 (1994). The device represents a variable rate alternative to the fixed rate Precision Flow Rate Tubing™, precluding the need to change tubing when a new rate is desired.


2.1 Indications for Use:

The Precision Flow Rate Controller™ is intended for use as an element of Freedom Series Syringe Infusion Systems in the administration of 3 centipoise viscosity infusion fluids to be delivered to the patient in a precise manner over a specified period of time. Use only with KORU Medical Systems' Freedom DynEQ™ Syringe Infusion Pumps.

2.2 Contraindications:

The Precision Flow Rate Controller™ is contraindicated for use with Gravity Infusion Systems and any non-KORU Infusion Systems.

3. Illustration of Device with Descriptions: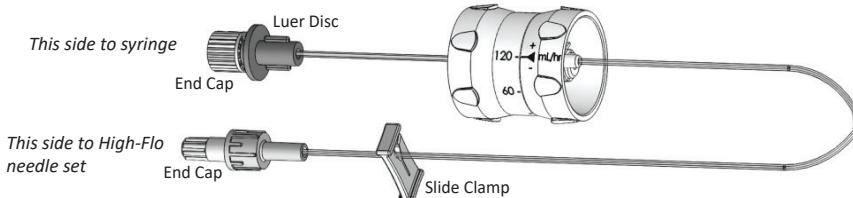

4. Instructions:

4.1 Pre-Set the Pump:

Ensure that the infusion pump you are using is properly configured to accept the syringe. (For more information, including the recommended syringes, see FREEDOM60® or FreedomEdge® Instructions for Use.)

4.2 Priming:

- a. Remove the cap from the luer disc end of the flow rate controller using sterile technique and care not to contaminate the luer connectors of the product, and connect it to a pre-filled syringe; remove the cap from the other end of the flow rate controller and using sterile technique, connect to the needle set.

1. Tighten connections completely. If leaking occurs, be sure that both the luer disc and needle set connections are tight.


b. Prime the controller by hand or with the pump.

1. If priming by hand, set flow rate controller to 300 ml/hr,

a. Grasp the syringe and gently press down with steady pressure.

b. Try to stop the flow about 2 inches short of the

needle(s). Rotate controller to the OFF position by aligning the arrow on the right side with the dot marked "OFF" on the left. To minimize site irritation, it is recommended to insert the needles dry.

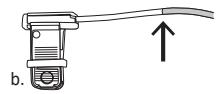


2. If priming by pump, set flow controller to 300ml/hr,

a. Make sure the pump is in the OFF position (if using the FREEDOM60®) or open (if using the FreedomEdge®).

b. Turn the pump ON (FREEDOM60®) or if using the FreedomEdge®, simply close the top lid to prime (fill) the tubing.

c. Try to stop the flow about 2 inches short of the needle(s) by turning the pump OFF (for FREEDOM60®) or opening the top lid (for FreedomEdge®). Rotate controller to the OFF position by aligning the arrow on the right side with the dot marked "OFF" on the left. To minimize site irritation, it is recommended to insert the needles dry.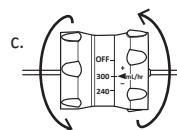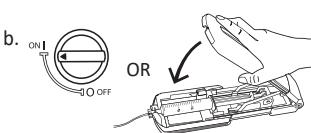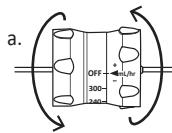

Note: The Precision Flow Rate Controller™ is designed to work with KORU HIGH-Flo 24 gauge single needle set. However, if two needles are being used, the controller total flow rate (combined) will be slightly higher than marked. Maximum flow rate with two needles at the 300ml/hr setting is approximately 360ml/hr (180ml/hr/site).



4.3 Infusion:

a. From the OFF position, rotate controller to the setting determined by your healthcare professional.

b. Begin infusion by turning the pump ON (if using the FREEDOM60®) or by closing the top lid (if using the FreedomEdge®).

c. Select flow rate as directed.

d. Use slide clamp to stop flow in case of emergency.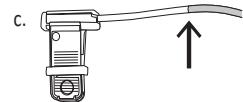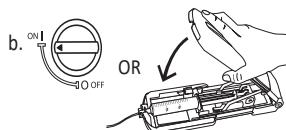


4.4 At End of Infusion:

a. Once the infusion is complete, turn the pump to the OFF position (FREEDOM60®) or open the top lid (FreedomEdge®).

b. One at a time, hold each needle in place and peel back any adhesive or dressing around it. Pull the wings back and remove each needle straight out opposite the direction you inserted it.

c. Discard all sharps, tubing and biologics as instructed by healthcare provider.

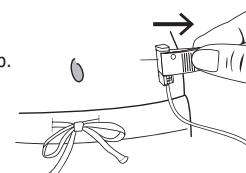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The Precision Flow Rate Controller™ is a trademark of KORU Medical Systems and are compliant with Medical Device Directive 93/42/EEC. KORU Medical Systems is ISO 13485 certified. ©2020 KORU Medical Systems; All Rights Reserved. FRC_IFU_342313vE